

OnGuard

One Size Fits All Face Shield

①

Shield

Headband

FOAM

What you'll need

If shield isn't clear
remove protection film

②

Insert headband into the inside slots on the shield (1)

Gently compress each side of stabilizer to fit through slots

Weave the band through the second slot (2)

③

Align Stabilizers between the slots of the face shield

Ensure both foam are spaced for comfort

④

Place on forehead and attach straps for a comfortable fit

Care instructions:

Gently wash with warm water and mild soap or Isopropyl alcohol 70%.

DO NOT clean with ammonia or alcohol based cleaning products